SanTRAL[®] Plus SMART TTU 2 Smart Retrofit Kit

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Contents

1. General safety.....	18
2. Safety when operating the device	18
3. Intended use.....	19
4. Scope of delivery.....	20
5. Technical description.....	20
6. Assembly	21
7. LED display.....	24
8. Maintenance and care.....	25
9. Battery	25
10. Risks	26
11. Rectifying faults	27
12. Disposal	28
13. Technical data	28

Notes on using the instructions

The following symbols are used in the instructions:

Safety notice:
Non-observance can result in personal injury or equipment damage.

Learn more:
SanTRAL® Plus SMART
Scan QR code or enter
<https://www.ophardt.com/en/santral-plus/smart>
in the browser

1. General safety

-  Repairs to the device may only be carried out by OPHARDT employees or OPHARDT-authorized persons. Serious hazards may ensue if repairs are not carried out correctly. In the event of repairs, contact OPHARDT customer service.
-  Mount the device in a area that is dry and protected from splashes.
-  This device is not a toy. Keep and use the device out of reach of children.
-  Only operate the device with leak-proof alkaline batteries of the same manufacturer and type and also with the same charge level. Use only AA batteries.
-  Do not use the device with rechargeable batteries.
-  Stop using the device altogether if the batteries have leaked in the compartment.
-  Dispose of discharged batteries at dedicated collection points.
-  Never use force.
-  There is absolutely no need to dismantle the device. It has no repairable parts.
-  Never expose the device to thermal radiation or the effects of heat.

2. Safety when operating the device

-  Protect the device against moisture. Do not spray the device to disinfect/clean it. Observe the cleaning instructions (item 10).
-  Never submerge the device in water.
-  Only fill the device with the relevant product.
-  There are no removable parts except for: battery box, battery
-  The device must not be cleaned with a water jet or high-pressure cleaner.
-  The device is not autoclavable and must not be cleaned in a dishwasher.

3. Intended use

The product is a retrofit module for SanTRAL® Plus TTU 2 toilet paper dispenser. It detects the fill level and battery charge fully automatically and sends these details to a higher-level evaluation system. The unit is designed exclusively for battery-powered operation. The unit may only be used inside buildings.

Environmental conditions:

- Never use the unit in explosive areas.
- Only use the unit in a low dust environment.
- Make sure that the unit is only used with a relative humidity of <90% (not condensing).
- Only use the unit at room temperature.

Correct handling of retrofit unit

To ensure that the unit functions optimally at all times, observe the following:

- Only use retrofit unit in connection with genuine SanTRAL® Plus TTU 2 toilet paper dispenser.
- Pay careful attention to the polarity when inserting batteries.
- The retrofit unit must not be operated near radiation-emitting or statically-charged objects.
- To safeguard the function, the retrofit unit including magnet must be installed correctly in the dispenser.


Attention!

OPHARDT will accept no liability for damage caused by the following:

- Damage to the device from mechanical influence and surges
- Modifications to the device without the express permission of OPHARDT
- Operation in a way other than intended

4. Scope of delivery

- Kit SR TTU 2
 1. Battery box inc. 4x AA batteries
 2. Electronics box
 3. Magnet
 4. Magnet holder
 5. Kit cover
 6. 2x nuts
- Assembly instructions

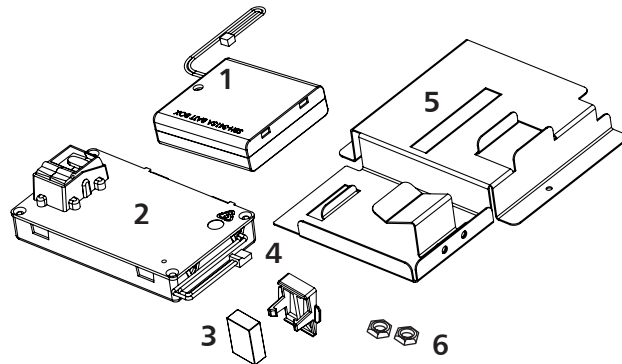5. Technical description

The retrofit unit continually measures the fill level of the toilet paper dispenser and detects other device information: battery charge, opening and closing of the housing and device restart (for example after battery replacement). This information is sent by the retrofit unit to a higher-level evaluation system by means of WiFi.

If the fill level is low, a yellow LED on top of the unit flashes and indicates to the user that the product will soon be empty and should be refilled.

If the battery charge is weak, a red LED on top of the unit flashes and indicates to the user that the batteries need to be replaced.

Device and WiFi settings can be configured over an NFC interface conveniently using the OIS app.

Further processing of the data and a detailed overview are facilitated with the Canary® software. The software is not included in the scope of delivery. For further details, please visit <https://www.ophardt.com/en/kanary> on the internet or simply scan the QR code.

6. Assembly

Preparation for installation

During installation, make sure that there are no radiation-emitting objects (X-ray machines, cellphones, ...) in the immediate vicinity. Electromagnetic interference and / or radiating electrical fields can affect the function.

Compatible with all TTU 2 dispensers. For simple installation of the smart retrofit kit, mount the TTU 2 dispenser on the wall.

To use all of the functions, the following conditions must be met:

- A reliable WiFi signal of IEEE 802.11 b/g/n standard covering the area of use must be available
- Data from the TTU 2 dispenser are collected in the message collector (OCG = Ophardt Communication Gateway). This can be installed locally or made available as an option by Ophardt Hygiene in the cloud. The data are then transferred to the Ophardt IoT cloud platform.
- The smart retrofit kit must be configured for use in the WiFi network.
- For analysis of the data, an internet-ready end device is required (e.g. PC, laptop, tablet, smartphone).

Commissioning is carried out by OPHARDT Hygiene employees or persons authorised by OPHARDT Hygiene.

Required tools

- Flat head screwdriver (e.g. blade width 1x4 mm)
- Pliers (e.g. engineers pliers)
- Socket insert with ratchet and extension (size 5.5/M3)

Assembly


- **Warning:** OPHARDT will accept no liability for damage caused by excess force or incorrect assembly.
- **Warning:** Risk of injury if assembly is not carried out as detailed. Assembly may only be carried out by qualified personnel who have read and understood the operating instructions.


For assembly of the TTU 2 smart retrofit kit, perform the following steps in the specified order:

Assembly (continued)


1. Open dispenser with key. Then raise the hood, and swivel it forwards.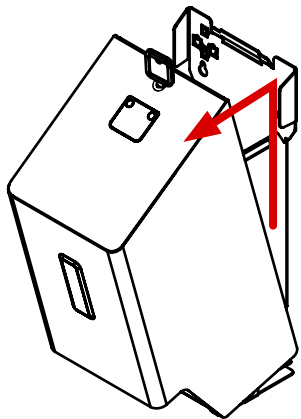


2. Carefully remove dummy plugs with a flat head screwdriver.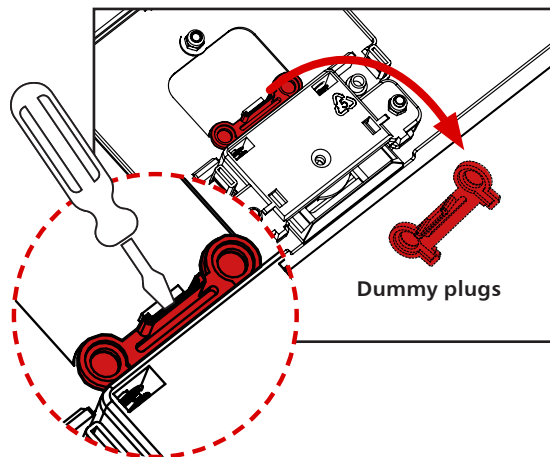

5. Place kit cover over electronics box and secure with two M3 nuts.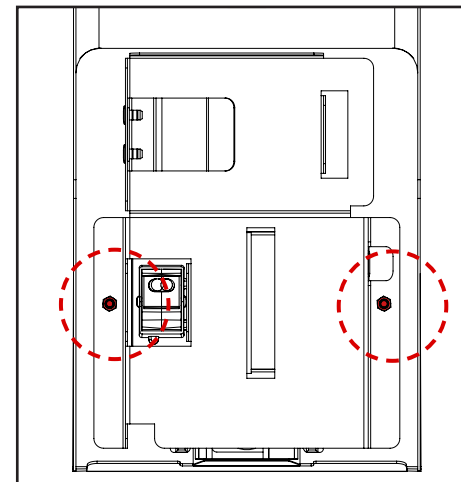



6. Fit battery box in holder on kit cover.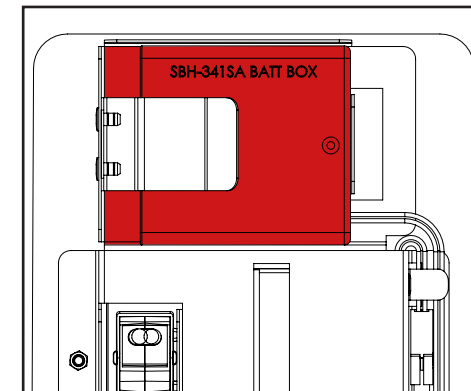


3. Click electronics box into dispenser.


4. Bend cable retainer outwards using a flat head screwdriver.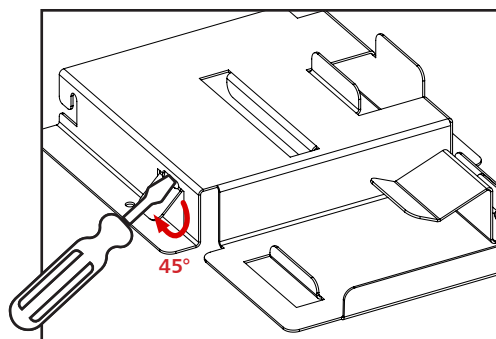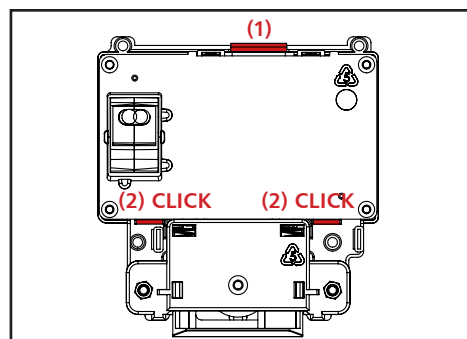

7. Join cables of electronics box to cables of battery box and guide through cable holder as per diagram.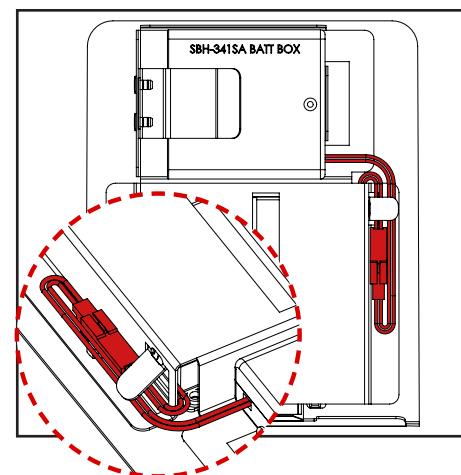

8. Press the two tabs of the magnet holder (3) in slightly and insert into the inside of the rear panel. Then insert the magnet (4) into the holder from the top front..

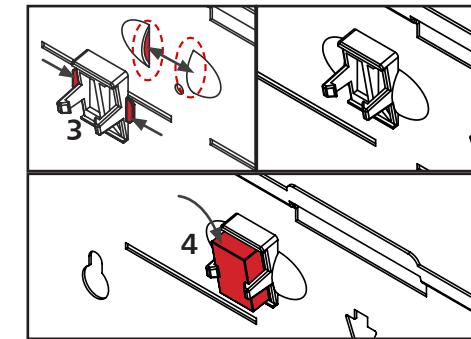Assembly (continued)

7. LED display

All SanTRAL® Plus smart devices are equipped with two LEDs:

Yellow fill level LED

Red battery LED

Battery charge

The flashing pattern is repeated every 30 seconds.

Flashes 1x = batteries are weak. Please replace batteries.

Flashes 3x = batteries are discharged. The device will soon cease to function. Please replace batteries.

Fill level

The flashing pattern is repeated every 10 seconds.

Flashes 1x = reminder to refill toilet paper dispenser.

Flashes 2x = toilet paper dispenser is nearly empty.

Flashes 3x = toilet paper dispenser is empty.

Opening/closing the housing

Flashes 1x = housing was opened.

Flashes 2x = housing was closed.

8. Maintenance and care

Cleaning and care

Please note that this high quality product requires care and attention. The device itself is maintenance-free with the exception of the batteries.

The recommendations and/or regulations of the responsible national institution - in Germany the Commission for Hospital Hygiene and Infection Prevention (KRINKO) at the Robert Koch Institute - must be observed in regard to cleaning and disinfection. The requirements in the institution's hygiene plan must also be fulfilled. The device is only cleaned from the outside.

The device may only be cleaned on the outside. Wipe down the device with a damp cloth using just water. Use plastic-compatible disinfectant cloths (wet wipes) to disinfect.

In the event of soiling, carefully clean the lens of the optical fill level sensor with a soft, damp cloth.

Never dismantle the device for cleaning purposes.

- Do not submerge the device in water or expose it to splashes.
- The device is not autoclavable and must not be cleaned in a dishwasher.

9. Battery

If the red battery LED flashes every 30 seconds, the batteries must be replaced.

Warning

Major hazards may ensue if the batteries are not changed correctly. This device contains alkaline batteries that cannot be recharged.


- Never connect this battery to a charger.
- The battery must not be short circuited.
- Dispose of discharged batteries at dedicated collection points.

Replacement battery

Only use leak-proof alkaline batteries of the same manufacturer and type and also with the same charge level. Use only AA batteries.

Battery replacement

1. Open dispenser with key. Then raise the hood, and swivel it forwards. (See 6. Assembly, step 1)
2. Remove battery box from holder and disconnect cable from electronics box. Press the locking mechanism of the connector to do this. (See figures 2.a and 2.b)
3. Unscrew and open battery box. Replace 4 AA alkaline batteries with new ones. Observe correct polarity. Close and screw on battery box.

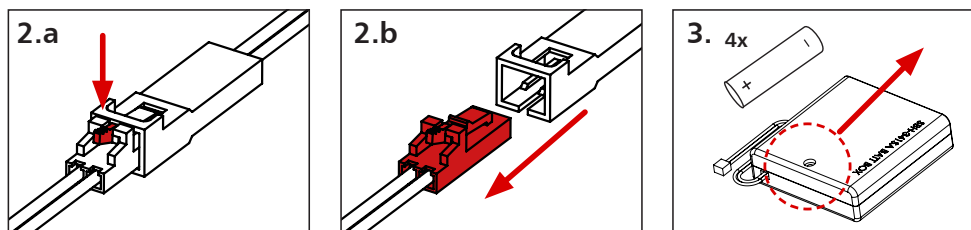4. Push battery box into holder on kit cover. (See 6. Assembly, step 6)
5. Join cables of electronics box to cables of battery box and guide through cable holder. (See 6. Assembly, step 7)
Close dispenser.

10. Risks

Risks

When handling the smart retrofit module, take note of the main differences compared with the smart-ready SanTRAL® Plus product series.

- The retrofit module contains electronics and batteries
- The retrofit unit is not autoclavable

Other risks

- Do not carry out repairs on the device.
- The battery must not be charged.
- The battery must not be short circuited.
- The battery must not be damaged.
- The magnet can burst if struck.

11. Rectifying faults

Some suggested solutions for rectifying faults can be gleaned from the adjacent table. Feel free to contact our service department as well: customercare@ophardt.com.

Characteristics	Possible cause	Solution
Fill level value fluctuates between two values.	Lens of optical fill level sensor is soiled.	Carefully clean lens of optical fill level sensor.
Fill level remains at a certain percentage. No new fill levels detected.	Device is no longer capable of transmitting data.	Check whether fill level LED still functions. If yes, sensor is working but data are not being transmitted. Check WiFi connection of device.
	Batteries are discharged.	Change the batteries.
	Device no longer carries out measurements.	Housing is not closed properly. Check whether LEDs flash when housing is closed.
	Lens of optical fill level sensor is soiled.	Carefully clean lens of optical fill level sensor.
Data not visible.	Note that data transfer may differ depending on the device configuration.	Check configuration of device.
	WiFi not available.	Check WiFi connection.
	Incorrect configuration of WiFi and device	Check configuration and if necessary rectify.
Device cannot communicate with message collector	Message collector not running.	Start message collector
	Server configuration of device is faulty.	Configure device.
	Message collector has no internet connection.	Connect to internet.
	Proxy settings of message collector not adapted.	Adapt proxy settings.
	A previous attempt at transmission failed. Device assumes a faulty configuration or network problem.	Device starts waiting phase (standard: 8 hours). A new transmission attempt is not normally possible until after this phase. To start an earlier transmission, open and close the housing of the device.

12. Disposal

Packaging

Treat the packaging material as recyclable waste and dispose of accordingly.

Device

When the device has reached its end-of-life, observe the correct disposal regulations by consulting a recycling centre or specialist.

Battery

The device contains alkaline batteries that must be disposed of correctly. Batteries must not be disposed of through domestic waste system.

13. Technical data

Housing:	Plastic: ABS (Acrylonitrile Butadiene Styrene), polycarbonate
Inner part:	PCB with sensors and WiFi module 802.11 b/g/n (WPA, WPA2 and WPA Enterprise), NFC module
Firmware:	Updates and "over-the-air" (OTA) configuration
Voltage:	6.0 V
Battery:	4x AA alkaline batteries 1.5 V, replaceable, cannot be recharged
Protection class:	IP32
Conformity	RoHS compliant
CE marking:	EN 300328 integration test WiFi EN 300330 integration test NFC ETSI EN 62311.2008 EMF ETSI EN 301489-1 EMC

OPHARDT HYGIENE-TECHNIK GmbH
Lindenau 27 • 47661 Issum • Germany
Phone +49 28 35 18-0 • Fax +49 28 35 18-77
Email customer care@ophardt.com • www.ophardt.com